

BIOWAVE
SMARTER PAIN BLOCKING TECHNOLOGY



Federal Supply
Schedule Contract #
V797D-70205

**DESIGNED
to BLOCK
PAIN**

**Rx
ONLY**

for Veterans

BIOWAVEHOME

BIOWAVEHOME



Immediate Pain Relief

Many patients report a 75% reduction in pain scores and their pain relief improves even further following treatment.

Reference: Double Blinded, Randomized, Controlled Study Comparing BioWave vs. TENS for the Treatment of Chronic Low Back Pain, Weill Cornell Medical College



Deep Smooth Sensation

BioWave treatments range in sensation from tingling to a deep, smooth muscle massage.



Opioid Reduction

Clinical studies and doctors report that use of BioWaveHOME allows patients to reduce consumption of opioids and other pain medication by up to 50%.

Reference: Randomized Controlled Study on the treatment of post-operative pain from TKA surgery, Hospital for Special Surgery, New York City



Long-Lasting Pain Relief

Patients who respond experience pain relief and improved function for up to 24 hours following the treatment. Multiple treatments may provide a cumulative benefit.

Reference: NY Giants White Paper - IRB approved study on treatment of Acute Sports Injuries on NY Giants football players June 2003-Dec 2005, 600 treatments on 160 players

BioWave Testimonials

“BioWave numbs the pain. There are days when I can’t walk and BioWave treatments help me get back on my feet.”

A. Snead



“My therapist and I were blown away by how well BioWave worked for me. My range of motion improved exponentially.”

W. Bethea

BioWave Testimonials

“BioWave reduced my pain significantly. I love the fact that I can manage my pain without medication.”

D. Norman



“I use the BioWave, and received it from the VA in Lebanon, PA. I was on opioids for 10 yrs and BioWave is a lifesaver.”

D. Pfeifer

Instant Pain relief at the touch of a button

BioWaveHOME delivers therapeutic electrical signals through skin directly to pain nerves to:

- ◆ Block the pain signal
- ◆ Increase range of motion and
- ◆ Reduce stiffness and muscle spasm




Works like Novocain, except BioWave blocks the pain signal electrically instead of chemically.




How is BioWaveHOME superior to TENS?

While both modalities use electrical stimulation to treat pain, that is where any similarity ends. TENS transmits stimulation Transcutaneously (across the surface of skin) which may act as a distraction (gate control theory) temporarily reducing the sensation of pain.

BioWaveHOME on the other hand transmits stimulation subcutaneously (beneath the surface of skin) directly to pain nerve fibers. The stimulation encompasses pain nerve fibers and stops the transmission of pain signals to the brain providing instant, long lasting pain relief.

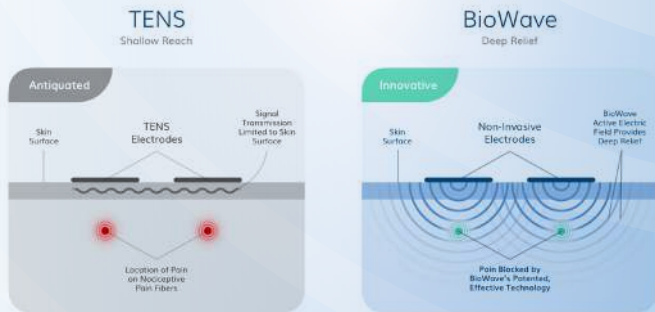

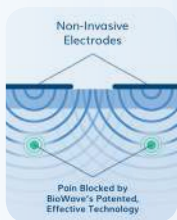

Blocking pain signals to the brain

BioWaveHOME transmits therapeutic electrical signals subcutaneously directly to pain nerve fibers. The active electrical field:

- ◆ Prevents action potential propagation along pain fibers
- ◆ Induces hypoesthesia on sensory fibers
- ◆ Increases blood flow through the volume of tissue being treated



BioWaveHOME is cleared by the FDA for use in treating chronic, acute or post-operative pain and available by prescription only. **Rx ONLY** **FDA**

How do I get BioWaveHOME from the VA?

Fill out the short contact form at www.BioWave.com/veterans or contact us at info@BioWave.com or 1-877-BioWave (1-877-246-9283).

How often can I have a treatment?

There is no limit to the number of 30-minute treatments that you can receive per day. The majority of patients report experiencing up to 24 hours of continued pain relief and functional improvement from a single 30-minute treatment. If you are experiencing a severe painful episode, then it may be beneficial to perform multiple treatments per day. There may be a cumulative benefit from multiple treatments, particularly if they are spaced from 1 to 6 hours apart. BioWaveHOME should be used on an as needed basis.

Is it right for me?

BioWaveHOME is a 30-minute treatment that can be used to relieve chronic, acute or post-operative pain in most locations on the body, including lumbar, thoracic and cervical spine, hips, groin, shoulders, elbows, wrists, hands, fingers, knees, ankles and feet.

Do I have to program the unit?

No! BioWaveHOME is easy to use. Turn it on and press the PLUS button. The unit delivers therapeutic electrical signals through skin to the surface of pain nerves that are optimized for blocking pain signals to the brain.

What level of activity may I perform after my procedure?

There are no limits to your activity after receiving a BioWaveHOME treatment. However, with less pain, patients should be careful not to perform strenuous activities that might have caused the pain in the first place.

What does the treatment feel like?

BioWaveHOME feels like a deep smooth pressure sensation, not the noxious twitchy sensation across the surface of skin like TENS.

To learn more ask your doctor
or visit [BioWave.com](https://www.BioWave.com).



CLEARED



ENDORSED BY
HEROES AND CHAMPIONS



MADE IN USA



BIO WAVE
SMARTER PAIN BLOCKING TECHNOLOGY

The information, including but not limited to, text, graphics, images and other material contained herein are for informational purposes only. It is not intended to be a substitute for medical billing advice, professional medical advice, diagnosis, or treatment. Any information contained herein are subject to a full examination of the medical studies, FDA filings and medical literature that underline the claims.

Learn more at BioWave.com

BioWave Corporation

8 Knight Street, Suite 201

Norwalk, CT 06851

www.BioWave.com

1.877.BioWave (246.9283)

rev 2 180503